



# Humasis FLU/RSV Combo


Providing a fast and accurate test result to aid early clinical diagnosis and to enable proper treatment



- ✓ Simple test procedure requiring minimal training
- ✓ Differentiate Flu A, Flu B and RSV with one sampling in 8 minutes


## Test Procedure

### Collection of sample

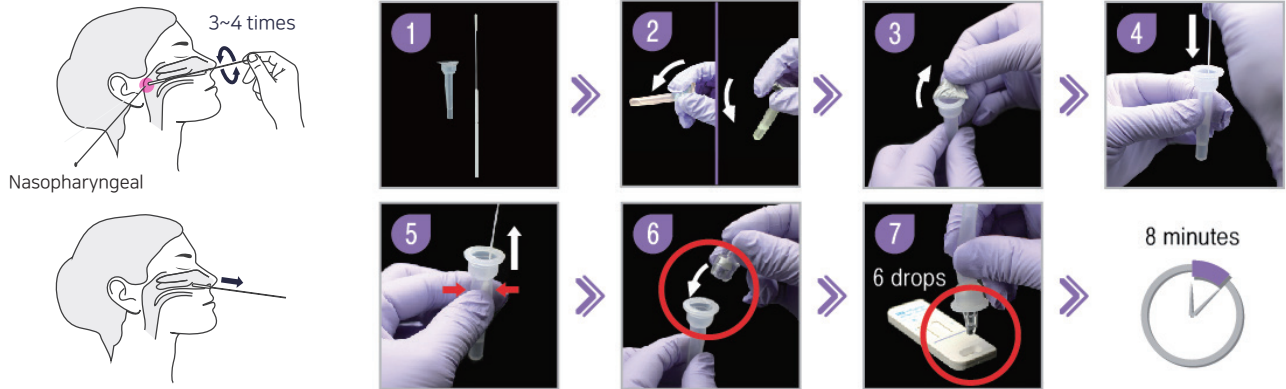


## Interpretation

### 1. Negative

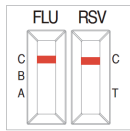


### 2. Positive


1) Positive for Influenza Type A

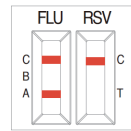







2) Positive for Influenza Type B

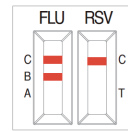

3) Positive for RSV

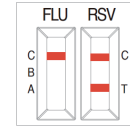

### 3. Invalid

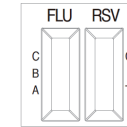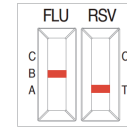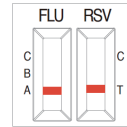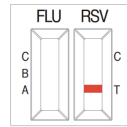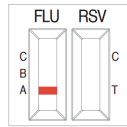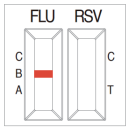

## Performance

Influenza A/B				Humasis FLU/RSV Combo		Commercial RDT	
Virus Culture/RT-PCR	POS	Type A	150	POS	NEG	POS	NEG
		Type B	50	47	3	135	15
		Sub Total	200	184	16	182	18
		NEG	200	2	198	2	198
Clinical Sensitivity				92%(184/200)		91%(182/200)	
Clinical Specificity				99%(198/200)		99%(198/200)	
Overall Accuracy				95.5%(382/400)		95%(380/400)	

RSV		Commercial RDT+RT-PCR		Total
		Positive	Negative	
Humasis FLU/RSV Combo	Positive	120	0	120
	Negative	0	203	203
Total		120	203	323
Clinical Sensitivity		100%(120/120)		
Clinical Specificity		100%(203/203)		
Overall Accuracy		100%(323/323)		

## Order Information

Cat. No.	Product Name	Package	Specimen	Shelf Life
AFRC-7030	Humasis FLU/RSV Combo	30 Tests/Kit	Nasopharyngeal Swab	24 months (Room Temperature)



### Humasis Co., Ltd.

Rm. 114, 502, 504, 604, 604-1, B03-1, B03-2,  
88, Jeonpa-ro, Dongan-gu, Anyang-si,  
Gyeonggi-do, 14042, Republic of Korea  
TEL:+82-31-8085-6200, FAX:+82-31-8085-6249  
Email:question@humasis.com  
www.humasis.com



### MT Promedt Consulting GmbH

Altenhofstr.80  
D-66386 St. Ingbert / Germany  
Tel: +49 6894 - 58 10 20 Fax: +49 6894 - 58 10 21  
Email: info@mt-procons.com  
www.mt-procons.com